

INSTRUCTIONS FOR USE



SEA AND  
SWIMMING  
FOR  
ALL

## A USEFULL AMPHIBIAN CHAIR

Entirely foldable, without any tool, SOFAO slides into a medium-size car boot.



The wheels are mounted on axes and are entirely removable. Then, they are easily assembled thanks to the pre-mounted split pin.

The front grip enables to move SOFAO around on loose terrains and allows to get easily in and out of the water.



## USE RECOMMANDATION



Do not forget to unfold the floaters as they ensure the floatability and stability.

## SOFAO IN SWIMMING-POOLS



Once the wheels assembled, you can drive SOFAO until the bathing site: its 80cm bottom width allows to pass through any swimming-pool front door



The armrests are foldable to enable an easy transfer from the wheelchair to SOFAO. Thanks to the adjustable backrest (5 positions), SOFAO can be put horizontally to change the user or shower him before entering the pool.



To fold and unfold SOFAO, it takes just around 20 seconds... without any tool

Its minimum size when folded enables an easy storage.



The safety belts are not compulsory, you can put them in the back for users who do not need them.



The guide, equipped with anti-slip soles, can put the user into the water by letting it roll along the pool wall. To get out, you just have to proceed the other way. For the heaviest passengers, we recommend one guide at the back and one guide in the pool.

## SWIMMINGPOOL OPTION

### FRONT ROLLER



You can also mount the front roller with brake onto the nose, thanks to a 13mm key, so that the guide can easily move around on poolside tiling or concrete surfaces. It also allows a simple entry into the pool.  
(See instructions for use)

### HEADREST

The headrest is adjustable and the backrest can be reclined in 5 positions so that anyone can find the suitable position. The backrest position is blocked thanks to a simple hand or foot pressure on the cable situated on the bottom back part of the backrest.



### STABILIZER-FLOATERS

The level of water can be regulated thanks to the 4 stabilizer-floaters. For a better use comfort, the floaters have to be positioned at shoulder level. It is still possible to adjust them once into the water.



## BEACH OPTION

### TIRE-SOFAO



Tire-Sofao enables to pull more easily on any kind of loose terrains. The front steer-wheel is connected to a towbar to enable hassle-free travel. It can be mounted on and off at front part of SOFAO.

### PAIR OF WHEELS



An extra pair wheel helps you to get across loose terrains, especially beach sand. They are mounted onto the original wheel axle.

## COMPLEMENTARY OPTION

### MULTIPOSITION HEAD - HUGGER



The head-hugger helps to maintain the users head during bathing. The positioning is adjustable depending to his height.

## USE



The user can move around in the water with small arm movement. If this is not possible, the guide can move SOFAO and the passenger thanks to the back handle or front grip.

The user always has to be under adult supervision.

## QUALITY

Thanks to its anodized aluminium and electropolish stainless steel structure, its Batyline® antifungal fabric: no particular maintenance is required. Just a spray-rinse at the end of each day out on the water, particularly on hinges.



## ENJOY YOUR BATHING TIME !

Sofao is a patented product and a registered trademark.

FERRIOL-MATRAT, ZI Charles Chana, 3 Rue de la Galerie, 42230 Roche-la-Molière (France)  
Tél : +33 (0)4-77-42-62-58 - Fax : 04-77-37-72-79 - Mail : [export@joeletteandco.com](mailto:export@joeletteandco.com)